



FEATURES
BROCHURE

enovis™

PECA[®]


BUNION IMPLANTS Ø3 / Ø4MM

BUNION SYSTEMS



KEY FEATURES

The PECA® Implant sytem provides a complete and versatile portfolio of beveled, fully threaded implants intended for the osteotomy fixation and arthrodesis of the foot. The Exact-T® Recess provides high precision in fluoro percutaneous implant positioning. Specialized soft-tissue sparing burrs and percutaneous instrumentation are used in combination with the PECA® bunion implant system to perform bone cuts and provide stable fixation.

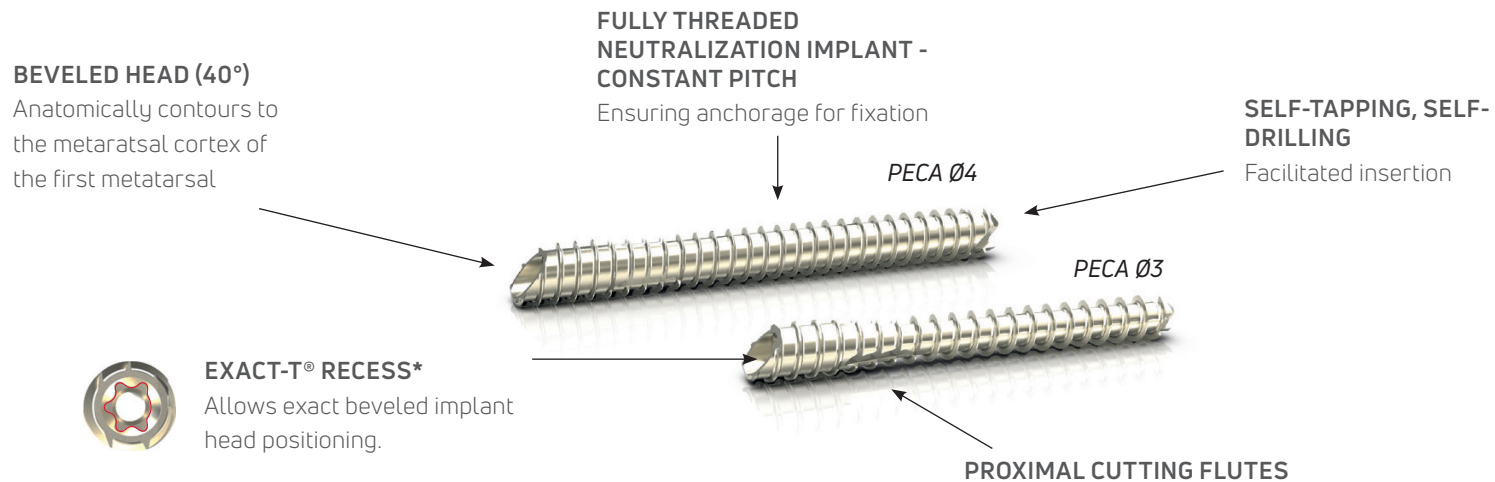

PECA® RANGE

Made of Titanium alloy (TA6V Eli anodised), PECA® Bunion implants are available in 3mm and 4mm diameters. Instrumentation is color-coded for convenient identification.

	Ø3mm	Ø4mm
DRIVER	EXACT-T8	EXACT-T10
LENGTH	16-48mm	26-60mm
K-WIRE	CoCr Ø1mm	CoCr Ø1.4mm
DRILL BIT	Ø2mm	Ø3.2mm

*Patent pending

INDICATIONS

The osteosynthesis screws are indicated for arthrosis, hallux valgus and other bone alignment defaults (pes cavus, flatfoot, calcaneal valgus, malalignment secondary to previous trauma).


EXAMPLE OF USE:

Hallux Valgus with Percutaneous Chevron and Akin osteotomies.



PECA® - PERCUTANEOUS INSTRUMENTATION

The PECA® set combines specific instrumentation for PECA® and Nexis® MIS implants, and percutaneous instrumentation including periosteal elevator, rasps, reduction device and beaver blade holder for fast, accurate and dedicated percutaneous surgery.

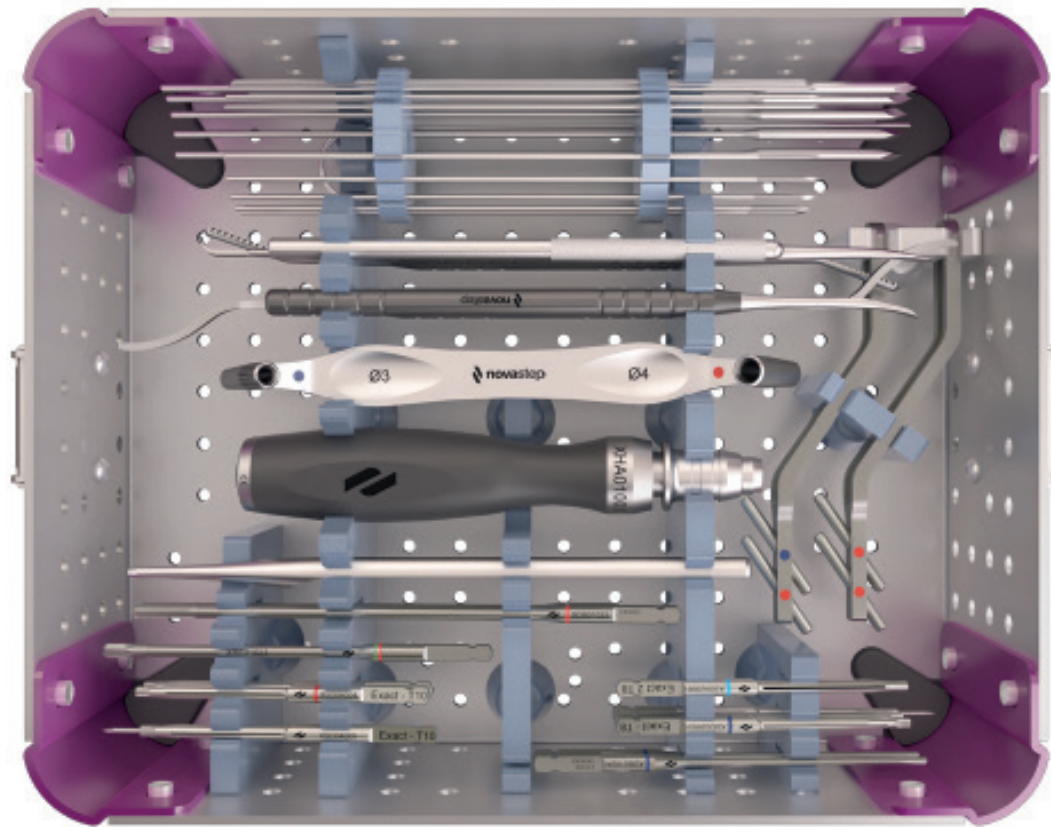

PERCUTANEOUS RASPS



PERIOSTEAL ELEVATOR DOUBLE TIP



PERIOSTEAL ELEVATOR SINGLE TIP



REDUCTION DEVICE DOUBLE TIP -
OPTIONAL



BEAVER HANDLE

PECA® - SPECIFIC INSTRUMENTATION

STERILE PERCUTANEOUS BURRS


Intelligently designed single use burrs offer precision bone resection and removal without violating soft tissue structures.



EXACT-T® TECHNOLOGY: PATENTED INNOVATION

Exact-T® (patent pending) facilitates correct placement of implant upon insertion.

EXACT-T® RECESS - PECA® IMPLANTS

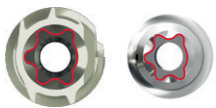

SPECIFIC

Easy indexing of the Exact-T® screwdriver tip allows exact driver positioning in one direction only

REFERENCES

PECA® BUNION IMPLANTS

LENGTH (mm)	PECA® Ø3mm	PECA® Ø4mm
16	PS020016	-
18	PS020018	-
20	PS020020	-
22	PS020022	-
24	PS020024	-
26	PS020026	PS050026
28	PS020028	PS050028
30	PS020030	PS050030
32	PS020032	PS050032
34	PS020034	PS050034
36	PS020036	PS050036
38	PS020038	PS050038
40	PS020040	PS050040
42	PS020042	PS050042
44	PS020044	PS050044
46	PS020046	PS050046
48	PS020048	PS050048
50	-	PS050050
52	-	PS050052
54	-	PS050054
56	-	PS050056
58	-	PS050058
60	-	PS050060

K-WIRES & REDUCTION WIRE

REFERENCE	DESCRIPTION
CKW02005	K-WIRE Ø 1.4 LG 150 TR/RD CRCO
CKW02004	K-WIRE Ø 1 LG 100 TR/RD CRCO
CKW03001	REDUCTION WIRE Ø 3.5

STERILE PERCUTANEOUS BURRS

REFERENCE	DESCRIPTION
CRE12008	SHANNON CORTA Ø2 LG 8
CRE12012	SHANNON RECTA Ø2 LG 12
CRE12212	SHANNON HELICAL Ø2 LG 12
CRE12222	SHANNON LONGA Ø2.2 LG 22
CRE13020	SHANNON LARGA Ø3 LG 20
CRE13030	SHANNON X-LARGA Ø3 LG 30
CRE23113	WEDGE Ø3.1 LG 13
CRE24113	WEDGE Ø4.1 LG 13

NOTES



T +33 (0) 2 99 33 86 50 F + 33 (0) 9 70 29 18 95

Legal manufacturer: Novastep® S.A.S
2 Allée Jacques Frimot | 35000 Rennes | France
contact-intfa@enovis.com
www.int.novastep.life

Copyright © 2024 Enovis Foot and Ankle

Carefully read the enclosed Instructions For Use (IFU) and all packaging label information. Devices: Implants: Class IIb-CE1639 / Instruments: Class I / Class Ir-CE1639 / Class IIa-CE1639.

REFERENCE: PECA-L-ED2-06-25-EN