

pecaplasty[®]

Percutaneous Bunion Correction

Forefoot Surgery



Brochure

—
Pecaplasty[®] targeting guide




- . **Easy positioning** on the foot
- . **Controlled translation** of the metatarsal head
- . **Accurate placement** of the K-Wires

 **novastep[®]**
Your foot & ankle company


pecaplasty[®]

Percutaneous Bunion Correction

The Pecaplasty[®] system allows for simple, precise and reproducible correction of Hallux Valgus in percutaneous surgery. With its easy positioning on the foot, the Pecaplasty[®] Targeting Guide allows controlled translation of the metatarsal head and accurate K-wires placement.

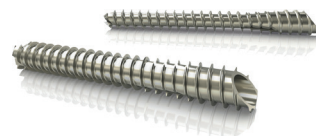



Keys features

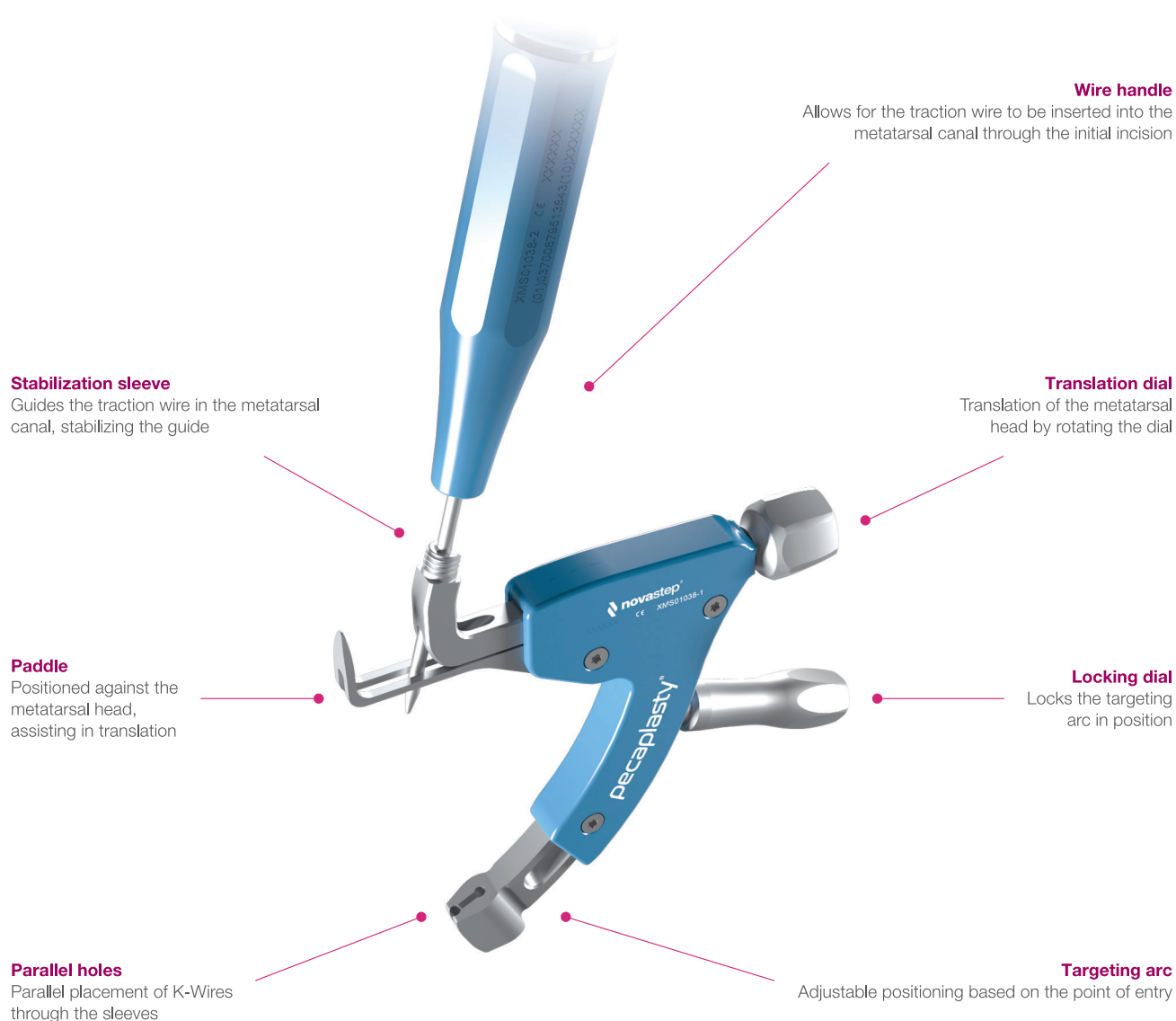

PATENT PENDING



Guided & reproducible procedure

1 Easy positioning on the foot

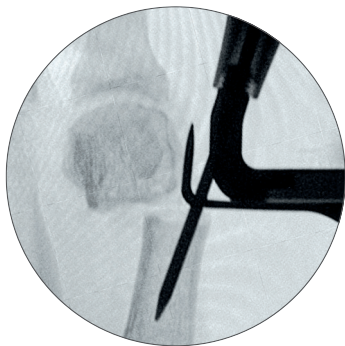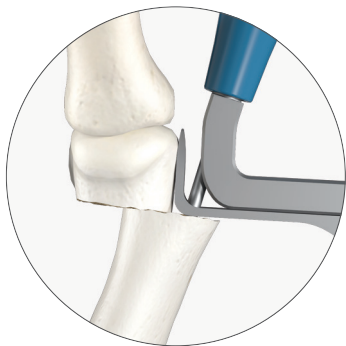



POSITION
the guide after a transverse osteotomy by inserting the paddle beneath the medial capsule.

2 Controlled translation of metatarsal head

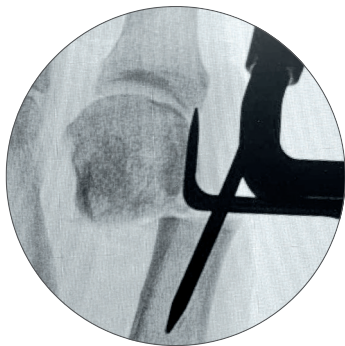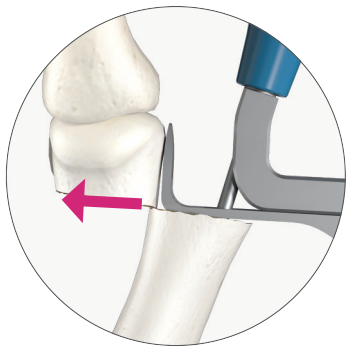

TRANSLATE
the metatarsal head using the stabilization sleeve.


3 Accurate placement of K-Wires

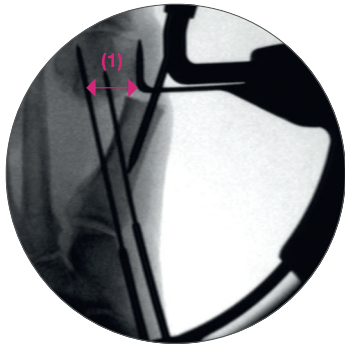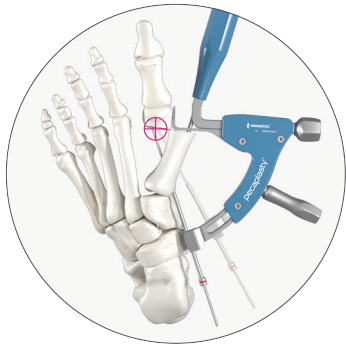

ADJUST
the arc around its center of rotation to allow K-Wires insertion. Insertion point must be as proximal as possible. Aiming point is fixed, always 14 mm (1) from the paddle.

References

Pecaplasty® targeting guide

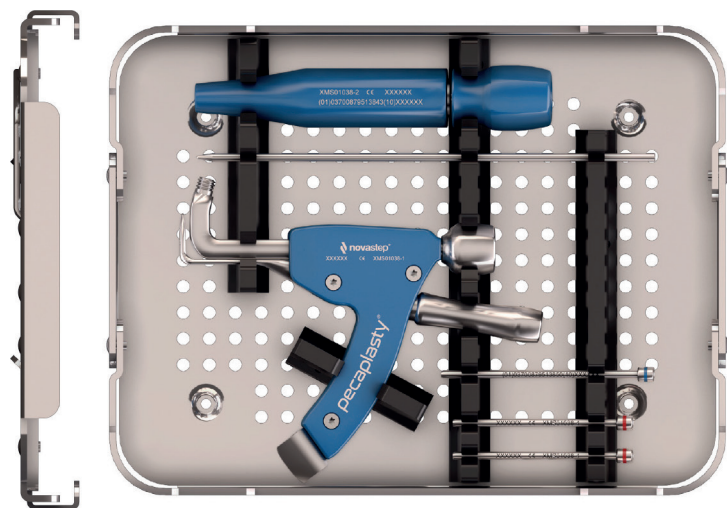

Reference	Designation	Qty	
XMS01038-1	Pecaplasty® targeting guide	1	
XMS01038-2	Pecaplasty® targeting guide wire handle	1	
SKW06001	Traction wire for Pecaplasty® targeting guide - Sterile	5	
XMS01038-3	Peca® Ø 3 K-Wire sleeve	1	
XMS01038-4	Peca® Ø 4 K-Wire sleeve	2	

Please Note:
Carefully read the enclosed Instructions For Use (IFU) and all packaging label information. Devices: Implants: Classe IIb-CE1639 / Instruments: Classe I / Classe IIa-CE1639.

Novastep:
2, Allée Jacques Frimot - 35000 RENNES - France
Tel: + 33 (0) 2 99 33 86 50 / Fax: + 33 (0) 9 70 29 18 95
contact@novastep-ortho.com / www.novastep-ortho.com

Reference: PECA-Tar-L-Ed1-12-21-EN